

Alerts911
MEDICAL ALERT SYSTEMS



Unlock the **EXPERIENCE** Now

Congratulations on your decision to use the **Care4Mom** App with your **On-the-Go Freedom!**

After downloading, just follow these instructions to access your new App.



Download the **Care4Mom** app
on Google Play (Android)



or Apple App Store (iPhone)




Where to start:




Create Your Account

Upon launching the app, click "Create Account" and fill in the requested information: your name, phone number and home address, your email address, and the password you would like to use.

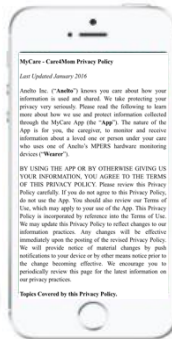

Accept Agreement

Read and accept agreement by pressing 'I Agree' on top right of phone screen.

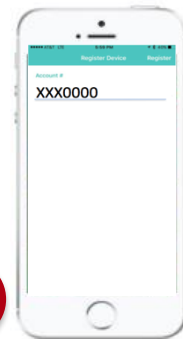


Account Number

Input the account number, the alpha-numeric code found on the box for the device, then click on Register. Note: If you are not the On-the-Go Freedom user, then you must be previously registered as an emergency contact on the account to gain access to the Care4Mom app. This should have been done at the time of setup. If you need assistance, please contact Alerts911.

How It All WORKS

Here is a brief overview on how to use your new app and its features.
Now all you have to do is open your app and you are ready to begin!


Home Screen

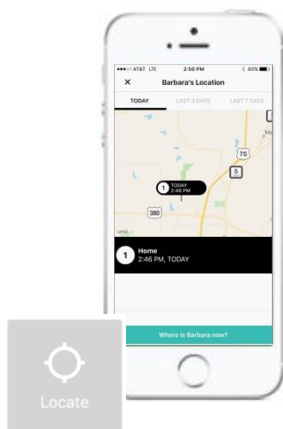


Press the **Locate** button to get instant GPS location for the On-the-Go Freedom. You will also be able to access the device location history for the previous 7 days.




Press the **Weather** button to get up to date weather conditions for wherever your On-the-Go Freedom is located.

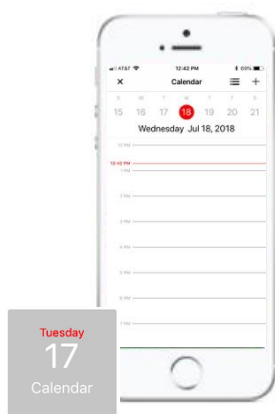

Press the **Calendar** button to schedule reminders or to check reminders previously scheduled.

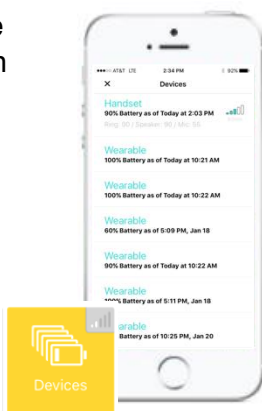

You will be able to view the battery life by pressing the **Devices** button in the app.

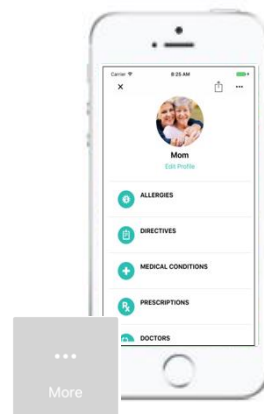

Use the **More** button to view and update your profile with information on allergies, medical conditions, medications, prescriptions, doctors and other caregivers on this app.

Did you know? You can manage multiple devices from this app if you have more than one person using an On-the-Go Freedom.