



Unlock the **EXPERIENCE** Now

Congratulations on your decision to use the **Care4Mom** App with your **On-the-go Freedom!**

After downloading, just follow these instructions to access your new App.



Download the **Care4Mom** app on Google Play (Android)



or Apple App Store (iPhone)




Where to start:



1

Create Your Account


Upon launching the app, click "Create Account" and fill in the requested information: your name, phone number and home address, your email address, and the password you would like to use.

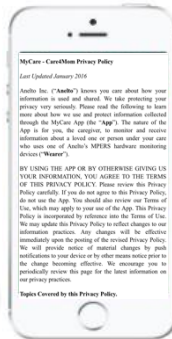

2

Accept Agreement

Read and accept agreement by pressing 'I Agree' on top right of phone screen.

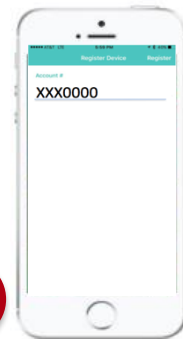

3

Account Number

Input the account number, the alphanumeric code found on the box for the device, then click on Register.

Note: If you are not the On-the-go Freedom user, then you must be previously registered as an emergency contact on the account to gain access to the Care4Mom app. This should have been done at the time of setup. If you need assistance, please contact Alerts911.

